Apex Medical S.L.

Elcano 9, 6º planta

Apex Medical Corp.

Manufacturing Facility: Apex Medical (Kunshan) Corp.

JiangSu Sheng, China

No. 1368, Zi Zhu Rd. Kunshan

Kai Fa Hi-Tech Kunshan City,

Print-2019/All rights reserved www.apexmedicalcorp.com

48008 Bilbao. Vizcaya. Spain

No.9, Min Sheng St., Tu-Cheng, New Taipei City, 23679, Taiwan

## Care for a Healthy Life

#### 1. Product Description

The Flexi-Air is a hybrid mattress replacement system that combines foam with air for effective pressure redistribution and increased levels of comfort whilst providing extra support to help prevent the patient from bottoming-out, and reduce the risk of them acquiring pressure damage.

The unique, patented holes-in-foam design is used as an aid to the prevention of pressure ulcers across the whole surface area of the patient's body. The Flexi-Air non-powered mattress so can offer the flexibity and versitiliy of a regular foam matress.

### NOTE, CAUTION AND WARNING STATEMENTS

NOTE: Indicates some tips.

CAUTION: Indicates correct operating or maintenance procedures in order to prevent damage to or destruction

of the equipment or other property WARNING: Calls attention to a potential danger that requires correct procedures or practices in order to prevent personal injury.

SYMBOLS	,		
	Refer to instruction manual/ booklet	Ø	Humidy Limitation
Ø	Do Not Dry Clean	X	Do Not Iron
0	Tumble Dry, Normal, Low Heat	95	Machine wash, regular/ normal, 95 degrees C (203 degrees F)
$\Rightarrow$	Do Not Bleach	ł	Temperature Limitation
***	Manufacturer	EC REP	Authorised representative in the European community
	Ø	Refer to instruction manual/booklet  Do Not Dry Clean  Tumble Dry, Normal, Low Heat  Do Not Bleach	Refer to instruction manual/booklet  Do Not Dry Clean  Tumble Dry, Normal, Low Heat  Do Not Bleach

#### 2. Intended Use

Thank you for choosing the Flexi-Air hybrid mattress from APEX. The Flexi-Air hybrid mattress is designed for prevention or treatment of skin breakdown and pressure ulcers, and as an aid to the prevention and treatment of pressure ulcer in patients. The Flexi-Air is an ideal matress to be used as a first line defence against the threat of pressure ulcers.

### 2.1 Preparation & Use

Warning: Consult your physician for choosing the most suitable mattress for your pressure ulcer prevention and treatment.

Caution: Before using the mattress, make sure that the bed has no rough edges over all the surfaces as they might damage the masstress.

Warning: Use a standard healthcare bed frame with this mattress, mattress platform and side rails (if used) must be properly sized to minimize any gaps to prevent the patient from being entrapped.

Caution: If the mattress is too firm upon arrival or after storage in a cold environment, let the mattress acclimatize to room temperature for 24 hours. If the problem persists, please contact the place of purchase or local distributors for assistance.

- Place the mattress on the bed with the APEX logo and foot image face up and near the foot end of the bed. Ensure the zips are closed and the zip flap completely covers the zip and the handles on both sides.
- Smooth out any creases on the mattress surface and it's ready for use. Ensure every part of the patient's body is being placed on the corresponding area of the mattress.

Warning: Do not smoke on the mattress and keep the mattress away from any source of heat when using. Warning: Mattress covers have passed skin sensitization, skin irritation and in-vitro cytotoxicity tests. However, if you suspect that you may have had or are having an allergic reaction, please consult a

physician immediately. Warning: The use of additional mattress covers, heavy duvets, sheepskins and similar items might reduce the effectiveness of the pressure reducing properties of the mattress or lead to increased body temperature which might cause skin maceration and are not recommended for use. Please

consult your physician for clinical advice. Caution: Prevent the mattress from getting cut or torn and other forms of damage when using patient transfer aids or other sharp equipment near or on the mattress.

#### 3. Maintenance & Cleaning

### 3.1 Inspection

Warning: Ensure the mattress is checked (foam, air cells and cover) for signs of damage, stains and tears after the discharge of each patient, after the end of a period of use or on a regular basis. If the foam core of the mattress is wet or badly stained, the mattress should be withdrawn from service. If the cover is badly torn or contaminated, replace the cover with a new one to prevent cross-infection.

Warning: Inspect the air hose beneath the cells for sign of damage on a regular basis to ensure effective product performance of the mattress is maintained. If the air hose is damaged, please contact the place of purchase or local distributors for servicing.

## 3.2 Cleaning

- 1. Light soilage: To reduce the risk of cross-infection and contamination, wipe down the cover with a damp cloth pre-soaked with warm water containing a mild detergent and rinse it with clean water using a single use, non-abrasive cloth.
- Heavy soilage: To reduce the risk of cross-infection and contamination, the cover should be cleaned with detergent and clean water then with 75% alcohol solution or a diluted solution of 0.1% sodium hypochlorite(1,000ppm), and rinse it with clean water using a single use, non-abrasive cloth.

Warning: Do not spray any fluids on the foam or soak the foam in any fluids as it will deteriorate the product performance of foam and accelerate the degeneration of foam.

Caution: Do not use Phenolic based products to clean the mattress.

Note: Top cover can also be completely removed for laundry at indicated temperature; however, it is recommended that the user still check with local policy to determine the time/temperature ratio

Cover Material : Stretch

**Drying:** After cleaning, ensure the mattress is air dried thoroughly or tumble dried on a low heat setting before refitting to the mattress; humid environments have been shown to lead to bacterial growth and might accelerate skin maceration on the patient's body.

Caution: Dry the mattress out of direct sunlight or dusty areas.

Warning: Ensure the cut-out of three triangle holes on the side bolster foam being placed towards the head pillow area when remaking the mattress.

## 4. After Use

4.1 Storage: It is recommended to store the matress in a clean and dry, environment, free from sharp edges to avoid any possible damage. Store the mattress with the cover within a protective cover/bag; never place

Warning: Store the mattress out of direct sunlight or dusty areas and keep it away from other heat sources to prolong the service life of the mattress.

**4.2** Transport: Be careful when handling the mattress to make sure no damage occurs to the mattress. The mattress can be lifted/carried by holding the handle bars on the sides.

Warning: Keep the mattress away from sharp edges or equipment and abrasive surfaces when

4.3 Re-Use: It is recommended a mattress cleaning record is kept to ensure the matress is cleaned regularly. Before each time of reuse, clean the mattress in accordance with facility's cleaning protocol, read the instructions given in section 3 in this manual for reference.

## 5. Expected Service Life

transporting

The product is intended to offer safe and reliable operation for 5 years when used or installed in accordance with the instructions set out in this manual and provided by APEX Medical Corp. APEX recommends that the product be inspected and serviced by authorized technicians if there is any sign of damage or concern with the product or its function.

Caution: The service life of the product can be reduced by extreme or incorrect usage.

## 6. Technical specifications

Mat	tress		Specification	
Dimension (L x W x H)		200 X 80 X 20 cm 78.7" x 31.5" x 8"	200 X 85 X 20 cm 78.7" x 33.5" x 8"	200 X 90 X 20 cm 78.7" x 35.4" x 8"
Weight		16 kg (35.2 lb)	18 kg (39.6 lb)	20 kg (44.0 lb)
	Temperature	Operation: 10°C to 40°C (50°F to 104°F) Storage: -15°C to 50°C (5°F to 122°F) Shipping: -15°C to 70°C (5°F to 158°F)		
Environment	Humidity	Operation:10% to 90% R.H. (non-condensing) Storage: 10% to 90% R.H. (non-condensing) Shipping: 10% to 90% R.H. (non-condensing)		
	Atmospheric Pressure	Operation:70 kPa to 10: Storage: 70 kPa to 101. Shipping: 70 kPa to 101	1.3 kPa 3 kPa	
Fire Safety Standards		BS 7177 crib 2, EN 597-18-2, BS 5852		
Max. Weight Capa	acity	200 kg (441.6 lb)		

- Consult the distributor for other technical documents.
- Please follow national regulations to dispose of the unit properly.
- As APEX is committed to continuous product improvement, the manufacturer reserves the right to change product specifications without prior notice.

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installato in conformità alle istruzioni stabilite nel presente manuale e fornite da APEX Medical Corp. In